

# *cryo*EMBEDDER® Staining Procedure

## **Marsing Stat H & E Stain**

(Using Linear Stainer)

Cup 1	Acetone for 30 seconds
Cup 2	Empty
Cup 3	Gills Hematoxylin
Cup 4	Gills Hematoxylin
Cup 5	Warm Running Water
Cup 6	Warm Running Water
Cup 7	Diluted Eosin (1 part Eosin to 5 parts Alcohol)
Cup 8	100% Alcohol
Cup 9	100% Alcohol
Cup 10	100% Alcohol
Cup 11	100% Alcohol
Cup 12	100% Alcohol
Holding tank	Clearant

Clips or rack should be cleaned with 100% Alcohol before reusing to prevent clearant artifact